

FACE MASK

HOW TO WEAR CORRECTLY

These masks add essential protection for you and your customers. With an adjustable nose clip and breathable material for enhanced comfort, they also feature a three-layer design for your protection:

Non-woven layer, with technology that offers better barrier protection than cotton or polyester equivalents

Filter middle layer, providing dust prevention and good air permeability,

Inner Non-woven layer, which is splash-resistant and reduces the risk of contamination

HOW TO WEAR YOUR MASK CORRECTLY:



STEP ONE:

Wash your hands before touching your mask. Holding the elastics at either side of the mask and place them behind your ears



STEP TWO:

Pull the mask down ensuring that your mouth and nose are completely covered. Adjust the nose clip to a comfortable position. It should hug your nose securely

IMPORTANT:

Remember to wash your hands after removing your mask. Do not reuse. Make sure to replace it if it becomes damaged or damp.